

Application Guidelines




1. Initial screw placement should be determined by the surgeon. Drill a pilot hole using the 1.8mm drill bit at the desired location for brow suspension. No tapping is required.



2. Load the Endobrow Push screw with suture passer onto the Hex Head Driver. If desired, the screw may be threaded with the suture at this step.*



3. Position the screw into the pilot hole (perpendicular to the bone) and push straight down until the screw is fully seated. No twisting is required during insertion.

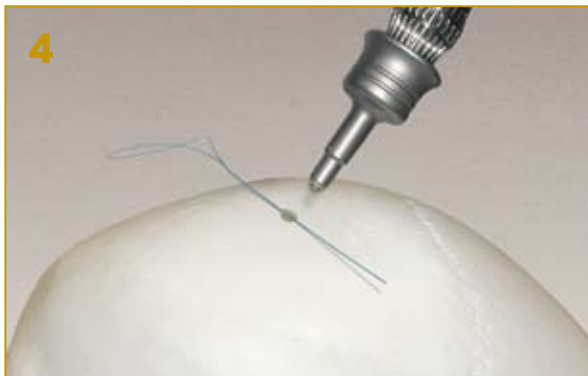

4. Disengage the hex head from the screw after insertion by gently turning the Hex Head Driver clockwise.



5. Suture is easily passed through the screw head utilizing the suture passer (if screw is not already threaded with suture) and is fixed to both the periosteal tissue and the LactoSorb® Endobrow Push Screw.

*LactoSorb® Endobrow Push Screw can accept 2-0 suture without a needle and 3-0 suture with a needle

BIOMET[®]
MICROFIXATION
Anticipate. Innovate.™